

3M Vetrap Bandage

Product Description:

A pet bandage staple. The elastic bandage that stretches and sticks only to itself. Lightweight, cool, comfortable, and will not absorb moisture. Vetrap is the first name in a flexible, self-adherent wrap that is strong and offers support without clips or fasteners. 2 inches x 5 yards. Assorted colors.

Instructions:

Apply 3M Vetrap Bandaging Tape at approximately 50% stretch. Applying too tightly may impair circulation. Apply last wrap with no tension; press firmly into place. Vetrap tape should be removed and reapplied daily. Vetrap tape should not be applied directly over open wounds

Precautions:

This product contains natural rubber latex which may cause allergic reactions in humans.